

AEROSOL DRUG DELIVERY
for
CANINE CHRONIC
BRONCHITIS



FREQUENTLY ASKED QUESTIONS

1. How can I purchase the AeroDawg* Canine Aerosol Chamber (CAC)?

Many veterinarians carry the AeroDawg* CAC in their clinic, but you may also purchase it through our website (click on the red 'Buy Now' button). Please speak with your dog's veterinarian before completing your purchase to ensure the device is suitable for your dog's condition.

2. How does the AeroDawg* CAC work?

The AeroDawg* CAC holds the puffer medication in the chamber allowing your dog to inhale using multiple breaths. In addition, the chamber separates the coarse aerosol particles (which tend to deposit in the nose and mouth) so the finer, more beneficial particles are delivered deep into the lungs where they are needed.

3. What medications can I use with the AeroDawg* CAC?

The AeroDawg* CAC is designed for use with most Metered Dose Inhalers (MDIs) or puffers as prescribed by your veterinarian. Trudell Medical International does not recommend specific medications.

4. Why would I use this over traditional oral steroids?

Tapering oral steroids over a two week program and moving to the inhaled steroids may offer a reduction in medication, providing cost effective therapy and likely reducing side-effects. Inhaled corticosteroids are the most effective drugs available to treat dogs with chronic bronchitis and should be considered the mainstay of chronic therapy.

5. Is there a reason why my dog could not use the AeroDawg* CAC?

We are not aware of any contraindications for the chamber in dogs. The principle behind the AeroDawg* CAC is based on aerosol drug delivery currently used in humans, horses and cats. However, the product should only be used if recommended by a veterinarian.



AeroDawg*
Canine Aerosol Chamber

www.aerodawg.com