AeroMask* Equine System ("ES")


INSTRUCTIONS FOR USE

The **AeroMask*** ES is intended for use with metered dose inhalers ("MDI") or wet nebulizers prescribed in equine care by veterinarians.

BEFORE USING THE **AeroMask*** ES, READ THESE INSTRUCTIONS CAREFULLY.


The **AeroMask*** ES with **AeroSpacer*** MDI Aerosol Holding Chamber ("AHC") selectively removes unwanted large aerosol particles that often deposit in the nasal passage (upper airway). The small particles are inhaled and deposit in the lower airways. This provides effective treatment while minimizing potential side effects of medications.

Components of the **AeroMask*** ES used with metered dose inhalers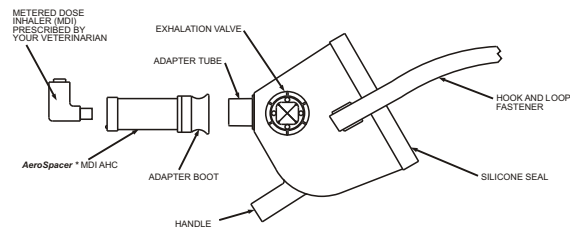


How to use the **AeroMask*** ES with an MDI

1. Insert the horse's nose into the **AeroMask*** ES.
2. Shake MDI and insert into **AeroSpacer*** MDI AHC
3. Attach **AeroSpacer*** MDI AHC to adapter port opening.
4. When exhalation valves close, actuate MDI.
5. For more than one puff, repeat steps 2-4 as prescribed, waiting 30 seconds between puffs

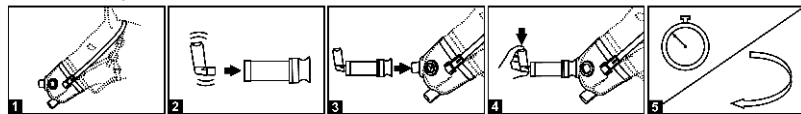
Components of the **AeroMask*** ES used with **AeroVet*** Aerosol Holding Chamber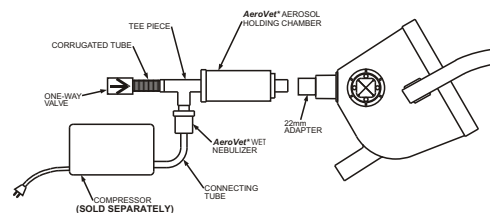

How to use the **AeroMask*** ES with the **AeroVet*** Aerosol Wet Nebulizer

1. Insert the horse's nose into the **AeroMask*** ES
2. Attach 22mm adapter (blue) to center port opening
3. Attach assembled nebulizer filled with prescribed solution to 22mm adapter
4. Turn on compressor
5. When "spitting" occurs in nebulizer cup, treatment is complete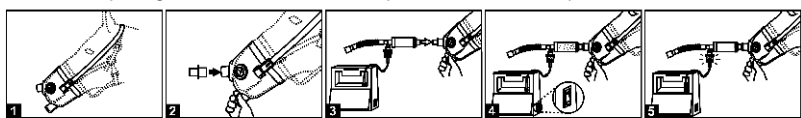

NOTES:

1. Please contact your veterinarian with questions about the **AeroMask*** ES.
2. Read all instructions provided with prescribed medications.
3. For MDI use, do not spray more than one puff at a time. Wait 30 seconds between each puff.
4. If the horse shows signs of breathing stress, while wearing the mask, simply remove one exhalation valve between puffs to relax the horse.
5. Before first use, you may remove the exhalation valves and permit the horse to become comfortable. The horse should accept the mask without hesitation. When the horse has become comfortable, attach both exhalation valves and proceed with the treatment.
6. Before using the **AeroMask*** ES, ensure both exhalation valves are securely locked into the mask.
7. The **AeroMask*** ES is available in three different sizes and can be used with MDIs, wet or ultrasonic nebulizers. For more information visit the website at www.aeromask.com
8. **AeroMask*** ES is intended for single horse use to prevent cross-contamination.

CLEANING INSTRUCTIONS:

The **AeroMask*** ES is made from a durable material and has three removable valves. With repeated use, residue may accumulate inside the **AeroMask*** ES and around the diaphragm valves. Over time, this may interfere with the effectiveness of the **AeroMask*** ES. Clean weekly, or between uses to avoid residue build-up. The wet nebulizer cup (when used) should be cleaned after each use. The metered dose inhaler should be cleaned according to the manufacturer's instructions.

To Clean the **AeroMask*** ES and Accessories


1. Remove the silicone seal, straps and one-way valves from the **AeroMask*** ES shell. Remove the flexible blue adaptor from the **AeroSpacer*** MDI AHC. With the **AeroVet*** AHC, dismantle the nebulizer, tee piece and flexible blue back piece.
2. Soak the **AeroMask*** ES shell, silicone seal, one-way valves and accessory parts in a basin filled with mild detergent and warm water for 20 minutes. Agitate using a gentle motion and a cloth to remove any deposited debris.
3. Shake excess water out and allow to air dry.
4. Re-assemble according to illustration.

Notes:

1. Do not heat sterilize. Only cold sterilize if disinfection is required.
2. Curled, hardened or damaged valves should be replaced.

This package contains:

- One complete **AeroMask*** ES
- Three spare/replacement exhalation valves
- One **AeroSpacer*** MDI AHC kit
- One **AeroVet*** Aerosol Holding Chamber and Nebulizer kit

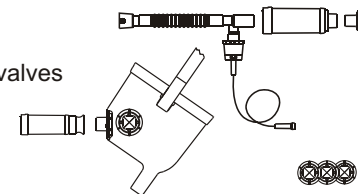Warranty

Trudell Medical International (TMI) warrants the **AeroMask*** Equine System ("ES") to be free from material defects in materials and workmanship for a period of six (6) months after purchase, provided the ES has been operated under conditions of normal use as described in the instruction sheet.

TMI will, at its option, replace, repair, or issue a credit for equipment or parts which are found to be defective. The defective ES components may be returned prepaid to our distributor after the customer has received approval from our distributor to return the components.

The foregoing shall be the sole and exclusive remedy of our distributor and its ultimate customers for TMI's liability of any kind including, without limitation, neglect, with respect to the ES. No other warranty or condition is expressed or implied, including, without limitation, any implied warranty of merchantability or fitness for a particular purpose. TMI shall not be liable for any incidental or consequential damages incurred or suffered by the purchaser or any other person or user of the ES as a result of a defect in any ES or component thereof.

WHAT IS NOT COVERED. This Warranty does not apply to any ES, or individual ES components, which have been altered in any way that, in TMI's judgement, adversely affects the performance of the ES, or which has been subjected to misuse, negligence, abuse or accident.

SHIPPING DAMAGE. The Warranty does not cover damage which may occur in shipment. If there is any damage in shipment, please contact the carrier or the individual who delivered the ES.

CHANGES TO EQUIPMENT. TMI, its dealers and associates reserve the right to make changes in equipment built and/or sold by them. These changes will not obligate TMI to make the same or similar changes to equipment previously built and/or sold by them.

FREIGHT. Freight-in is at the owner's expense. Freight-out is the responsibility of TMI or our distributor.

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