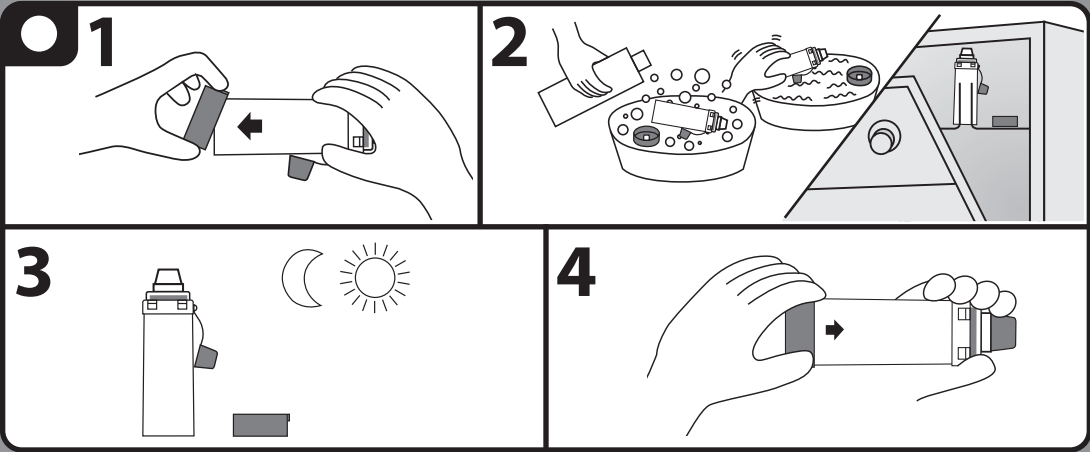
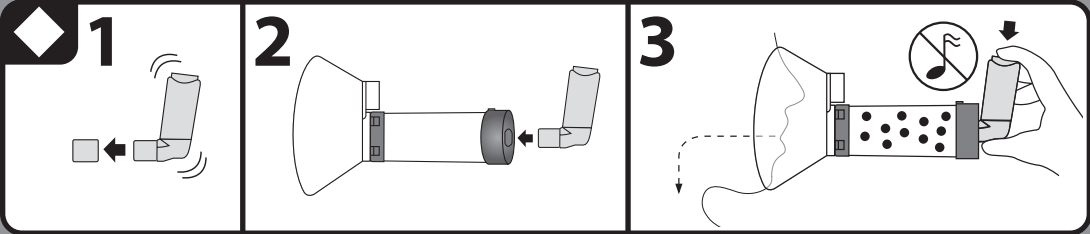
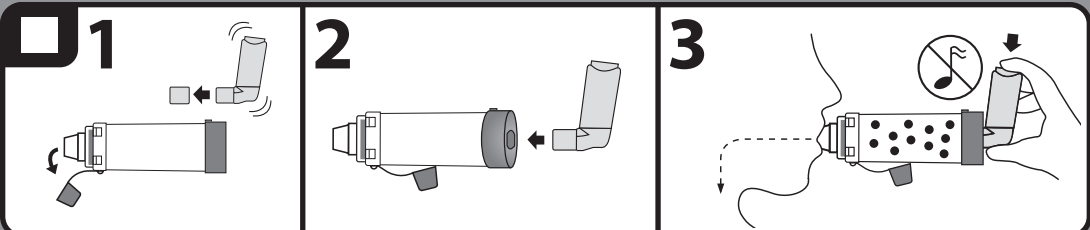
EN

INTENDED USE

This chamber is intended to be used along with a metered dose inhaler to deliver aerosol medication to your lungs as prescribed by your healthcare provider.

BEFORE USE

This chamber can be used directly out-of-package. Ensure these instructions and the instructions supplied with the inhaler have been read and are kept available at all times. Carefully look into the chamber. If there is any visible dust or other debris inside, then clean the chamber. Replace immediately if it is damaged or has missing parts.

1 MOUTHPIECE

1 Remove caps from inhaler and chamber. Shake the inhaler immediately before use as per the instructions supplied with it. **2** Insert the inhaler into the backpiece of the chamber. Put mouthpiece into mouth and close lips around it to ensure an effective seal. **3** Exhale and press the inhaler once at the beginning of inhalation. Inhale slowly and deeply through the chamber until a full breath has been taken. Hold your breath for 5-10 seconds before exhaling. **OR** Exhale and press the inhaler once at the beginning of a slow inhalation. Breathe in and out through the chamber for 2-3 breaths keeping lips sealed around chamber mouthpiece.

2 MASK

1 Remove cap from inhaler. Shake the inhaler immediately before use as per the instructions supplied with it. **2** Insert the inhaler into the backpiece of the chamber. Apply mask to face and ensure an effective seal. **3** Exhale and press the inhaler once at the beginning of a slow inhalation. Hold mask in place and breathe in and out through the chamber for 5-6 breaths.

3 Notes

- SLOW DOWN** if you hear the whistle sound, it means you are inhaling too quickly (on adult models).

- Administer one (1) puff at a time. Follow instructions supplied with the inhaler on how long to wait before repeating.

4 Cautions

- Do not use with Dry Powder Inhalers or Nebulizers. This chamber has not been designed to deliver these medications so you may not get the proper dose.
- Failure to follow instructions may affect medication delivery or cause inhalation issues.

5 Warning

- Do not leave the device unattended with children. Contains small parts that can cause choking.

6 CLEANING INSTRUCTIONS

This chamber can be used right out-of-package and then cleaned weekly.

1 Remove the backpiece only. Do not tamper with valve during cleaning or disassemble the product beyond what is recommended or damage may result. For mask product, do not remove mask.

2 Soak the parts for 15 minutes in a mild solution of liquid dish detergent and lukewarm clean water. Agitate gently. Rinse parts in clean water. **OR** Place parts in top rack of dishwasher. Ensure product is securely placed face up. Run the dishwasher on a normal or light cycle and avoid heated dry.

3 Shake out excess water from the parts and allow to air dry in a vertical position. Ensure parts are dry before reassembly.

4 To reassemble, press firmly to attach the backpiece. For mouthpiece models, the protective cap should always be placed on the mouthpiece when the product is not in use.

7 Notes

- Product should be replaced after 12 months of use.
- Do not share this medical device.
- Product is free of Bisphenol A, Pthalates, Latex, Lead and PVC.
- If you notice medication build-up in your chamber, wash the inside of the chamber gently with a soft cloth.
- Dishwashing with overly dirty dishes is not recommended.
- If cleaning in a dishwasher use a rinse aid.
- For cleaning of the inhaler, follow the instructions supplied with the inhaler.
- The chamber device can be disposed of with domestic waste unless this is prohibited by the disposal regulations prevailing in the respective countries.
- Put cap back on the mouthpiece and store in a clean, dry area when not in use.

8 Caution

- Do not boil or sterilize. Product may be permanently damaged if boiled, sterilized or cleaned in a dishwasher at a temperature above 70°C.

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