

AeroEclipse[®] XL

BAN[®] Nebulizer

OMBRA[®]

Table Top Compressor



Quick Start Guide



Place compressor on flat surface



Ensure valve on mouthpiece faces down



Unscrew top



Remove top




Place medication in cup



Reattach top




Gently hand tighten

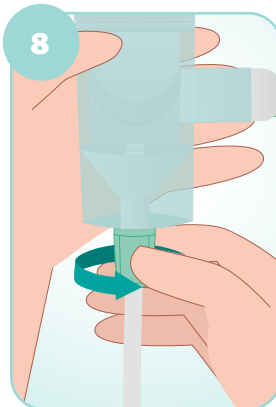

Attach tubing to bottom of nebulizer



Place nebulizer in cup holder and attach tubing to air inlet on compressor

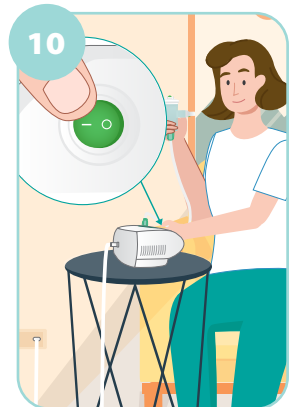

Plug in compressor and switch on




Place mouthpiece in mouth




Inhale slowly and deeply



Exhale normally

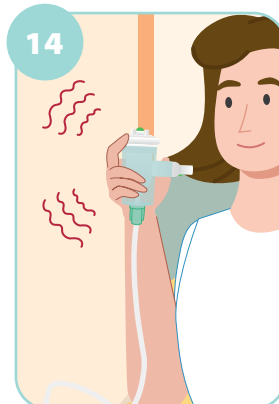

Continue until you hear a sputter sound

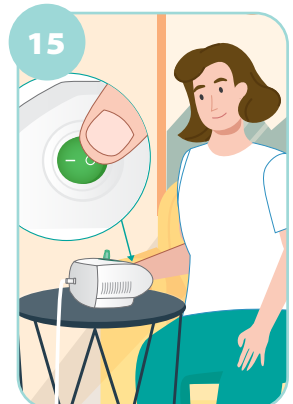

Switch off compressor and unplug. Disconnect nebulizer from compressor

AeroEclipse[®] XL

BAN[®] Nebulizer




Cleaning Methods




Soak in warm soapy water.



Rinse in clean water.

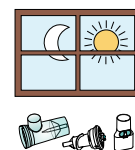

Air Dry thoroughly.

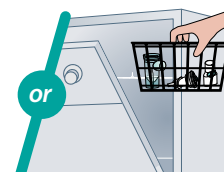

Dishwasher Safe top rack in basket.

Excluding tubing

Disinfection Methods



1 Boil
for 5 to 15 minutes.



2 Electronic Steam Disinfectant
See instructions for use.



3 Microwave Steam Bag
See instructions for use.

4 70% Isopropyl Alcohol

Soak for 5 minutes. Rinse thoroughly with sterile water.

5 1:50 Bleach Solution

Soak for 3 minutes. Rinse thoroughly with sterile water.

6 Milton[®], Dodie[®], Control III[®], Cidex[®]
See instructions for use.

7 1:3 Vinegar Solution

Soak for 1 hour. Rinse thoroughly with sterile water.

Excluding tubing

OMBRA[®]

Table Top Compressor



Cleaning Method

Unplug compressor from the electrical outlet.

Wipe case, switch, and handle with a damp cloth.



Disinfection Methods

Unplug compressor from the electrical outlet.



1 Wipe case, switch and handle with a tissue dipped in:

- 70% Isopropyl Alcohol
- 1:50 Bleach Solution
- 3M[®] SoluPrep[†] Solution

2 Wipe case, switch and handle with:

- PDI[†] Super Sani-Cloth[†] Germicidal Disposable Wipe

See instructions for use.

For Indications for Use and more detailed use instructions please refer to the Instructions for Use that accompanied your **AeroEclipse[®] XL BAN[®]** Nebulizer and your **Ombra[®]** Table Top Compressor (accessible at www.trudellmed.com)


Manufactured by:



**TRUDELL MEDICAL
INTERNATIONAL**

+1-519-455-4862

info@trudellmed.com trudellmed.com

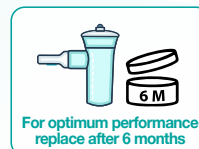

For optimum performance
replace after 6 months

LEARN
MORE

