


## Instructions for Use

1. Remove cap and shake inhaler
2. Insert inhaler into back of chamber and gently apply the mask to your cat's face
3. Press the inhaler to release the medication when the **Flow-Vu\*** Inhalation Indicator is moving toward your cat and hold the mask in place for 5–6 breaths

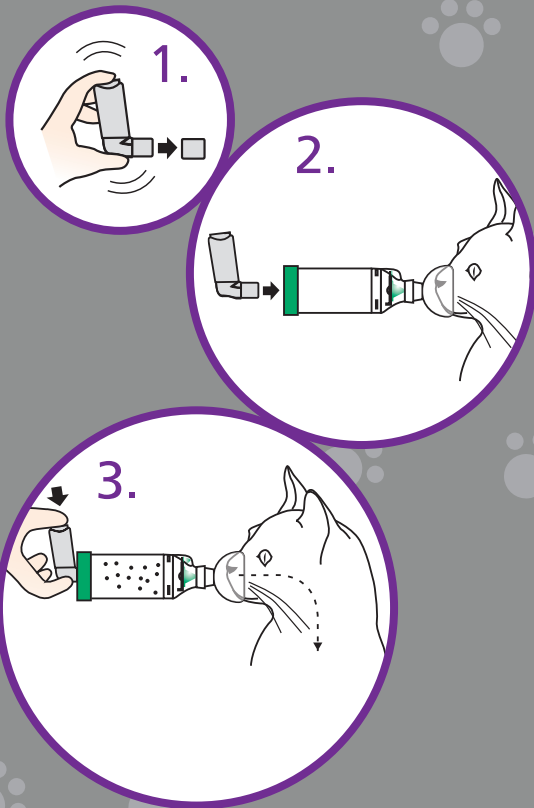

NOTE: The mask should cover your cat's nose and mouth but not the eyes.

## Tips and tricks

Here are some tips and tricks from some other **AeroKat\*** Chamber users.

- ▶ Reward your cat with treats before and/or after treatment
- ▶ Swaddle your cat in a towel or blanket
- ▶ Slowly introduce the **AeroKat\*** Chamber. Start with the mask only, getting your cat used to the feel of it
- ▶ Use the **Flow-Vu\*** Inhalation Indicator to help ensure your cat has inhaled their medication
- ▶ Reference the numerous videos on YouTube
- ▶ Be patient — don't give up!



725 Third Street, London,  
Ontario, Canada, N5V 5G4  
+1-519-455-7060 F: +1-519-455-7858  
1-800-465-3296  
[www.aerokat.com](http://www.aerokat.com)



101406001 Rev. D \*Trade-marks and registered trade-marks of Trudell Medical International. © TM 2015. All rights reserved.

# Does your cat suffer from asthma symptoms?





## Treatment

The **AeroKat\*** Feline Aerosol Chamber is designed to deliver Metered Dose Inhaler medication as recommended by a veterinarian to cats with feline asthma, chronic bronchitis or allergic rhinitis.

When the **AeroKat\*** Chamber is used with the inhaler it allows your cat to breathe normally and inhale the medication into their lungs, where it is needed.

Can be used with all types of Metered Dose Inhalers

If your cat experiences any of the following symptoms, they may be suffering from feline asthma:

- ▶ Frequent wheezing
- ▶ Persistent coughing
- ▶ Open mouth or labored breathing

Approximately one in 100 cats has asthma – perhaps more, as the symptoms are often confused with hairballs. Asthma usually affects cats between 2 and 8 years of age with Siamese and Himalayan breeds having a higher prevalence.

“Works like a charm — the mask fits *purr*-fectly over the nose/mouth and chamber holds in the meds to inhale. It even includes an indicator to show how many breaths are taken.

Highly recommend !!!”

Cat Owner

**AeroKat\*** 

Feline Aerosol Chamber



### **Flow-Vu\*** Inhalation Indicator

Takes the guesswork out of administering your cat's inhaler medication. The **Flow-Vu\*** Inhalation Indicator moves toward your cat as they inhale and will only move if there is a good seal.



### Inhalation Valve

Extra sensitive, low resistance valve is designed to respond to shallow, low volume breaths

### Anti-Static Chamber

Holds the medication allowing your cat time to inhale

### 2 Mask Sizes

Each device is shipped with 2 different sized facemasks. The masks are specifically designed to reduce irritation to your cat's fur.

