

AEROSOL DRUG DELIVERY for FELINE ASTHMA



Frequently Asked Questions

How can I purchase the *AeroKat** FAC?

Many veterinarians carry the *AeroKat** FAC in their clinic, but you may also purchase it through our website (www.aerokat.com). Please speak with your cat's veterinarian before purchasing in order to ensure the *AeroKat** FAC is suitable for your cat's condition.

How does the *AeroKat** FAC work?

The *AeroKat** FAC holds the puffer (corticosteroid or bronchodilator) medication in the chamber allowing your cat to inhale using multiple breaths. In addition, the chamber separates the coarse aerosol particles (which tend to deposit in the mouth and nose) so the finer, more beneficial particles are delivered deep into the lungs where they are needed.

What medications can I use with the *AeroKat** FAC?

The *AeroKat** FAC is designed for use with all commonly prescribed Metered Dose Inhalers (MDIs) or puffers as determined by your veterinarian.

Why would I use this over traditional oral steroids?

Traditional oral corticosteroids (pills, injection or syrup) have many serious potential side-effects for your cat including pancreatitis, insulin resistance (diabetes), polyuria, cystitis, inappropriate urination and behavioural changes.

Is there a reason that my cat could not use the *AeroKat** FAC?

There are no specific contraindications for the *AeroKat** FAC in cats. The principle behind the *AeroKat** FAC is based on aerosol (puffer) drug delivery currently used in humans, horses and dogs. The product should only be used if recommended by a veterinarian.



AeroKat  **t**
Feline Aerosol Chamber

www.aerokat.com