
ComfortSeal® Mask

with Elbow Adapter

(for use with AEROCLIPSE® Breath Actuated Nebulizers)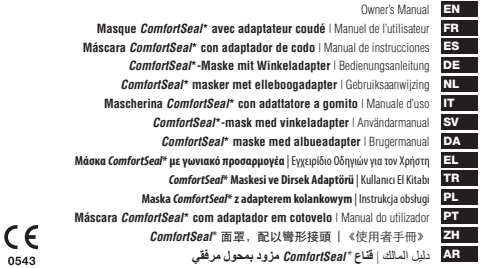

EN Owner's Manual
FR Manuel de l'utilisateur
ES Mascara ComfortSeal® con adaptador de codo | Manual de instrucciones
DE ComfortSeal®-Maske mit Winkeladapter | Bedienungsanleitung
NL ComfortSeal® masker met elleboogadapter | Gebruiksaanwijzing
IT Mascherina ComfortSeal® con adattatore a gomito | Manuale d'uso
SV ComfortSeal®-mask med albåreadapter | Användarmanual
DA ComfortSeal® maske med albueadapter | Brugermanual
EL Μάσκα ComfortSeal® με γωνιακό προσάρτημα | Εγχειρίδιο Οδηγών για τον Χρήστη
TR ComfortSeal® Maskesi ve Dizek Adaptörü | Kullanıcı El Kitabı
PL Maska ComfortSeal® z adapterem kolankowym | Instrukcja obsługi
PT Máscara ComfortSeal® com adaptador em cotovelo | Manual do utilizador
ZH Máscara ComfortSeal® 面罩，配以髁形接環 | 《使用者手冊》
AR دليل المستخدم، مرفقًا مع مرفق ComfortSeal®

Trudell Medical International®

725 Third Street
London, Ontario, Canada, N5W 5G4
Phone: +1-519-455-7060
Fax: +1-519-455-7858
customerservice@trudellmed.com
www.trudellmed.com

Canadian Hospital Distributor:
Trudell Medical Marketing Limited
754 Third Street,
London, Ontario, Canada N5W 5J7
trudell@trudell.com
www.trudell.com

Trudell Medical International Europe Ltd.
Elbow City Nottingham,
Pennington Street,
Nottingham, NG1 1GF, UK
customerservice@trudellmed.com
www.trudellmed.com

150568-001 Rev. D * Trade marks and registered trade marks of Trudell Medical International.
† Trade marks of respective companies. Copyright © Trudell Medical International 2012, 2013. All rights reserved.

FIGURE 1

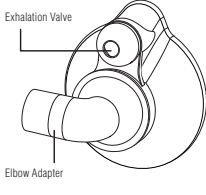
FIGURE 2

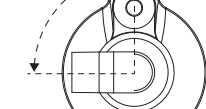
FIGURE 3

FIGURE 4

EN

Intended for use with the **AEROCLIPSE®** Breath Actuated Nebulizer (BAN).

INSTRUCTIONS FOR USE

Prior to each use, inspect the mask to make sure it is not damaged or missing parts. If you notice any damage or missing parts, replace it immediately.

- NOTE: When using the **AEROCLIPSE®** BAN with a **ComfortSeal®** Mask, ensure a secure seal all around the edge of the Mask. Even tiny gaps may allow ambient air to enter the Mask, significantly reducing the amount of medication delivered.
- Remove the **ComfortSeal®** Mask from its packaging and verify that the elbow adapter is firmly secured to the Mask (Figure 1).
- Ensure the Exhalation Valve located at the top of the Mask is intact.
- Rotate the Elbow Adapter so it is horizontal when the Mask is held in an upright position (Figure 2).
- Remove the mouthpiece from your **AEROCLIPSE®** BAN and attach the Elbow Adapter to the mouthpiece opening as shown in Figure 3.
- Ensure the mask maintains a good face seal prior to operating the **AEROCLIPSE®** BAN.
- Follow the operating instructions supplied with the **AEROCLIPSE®** BAN.

NOTE: We recommend using the **AEROCLIPSE®** BAN in Continuous Mode when using with a **ComfortSeal®** Mask.

CLEANING INSTRUCTIONS

Ensure you clean your **ComfortSeal®** Mask before first use and between each treatment.

- Remove Elbow Adapter from **ComfortSeal®** Mask and soak both parts for 15 minutes in a solution of liquid dish detergent and lukewarm clean water. Agitate gently.
- Rinse parts in clean, warm water.
- To dry, shake the parts to remove excess water. Allow to air dry thoroughly before reassembling.
- To reassemble, insert the Elbow Adapter into the opening on the **ComfortSeal®** Mask so it is horizontal when the mask is in an upright position.

DISINFECTING INSTRUCTIONS

The **ComfortSeal®** Mask should be disinfected at least every other treatment day.

- Prior to disinfection, follow Cleaning Instruction steps 1 and 2 above. Do not dry or reassemble the Mask.
- The **ComfortSeal®** Mask parts (mask and elbow adapter) should be soaked in 1:50 bleach to water solution (approximately 1

tablespoon bleach in 3 cups water) for 3 minutes. You may substitute a respiratory equipment disinfectant such as Milton®, Doodle® or Control III® for the bleach solution by following the manufacturer's instructions for use.

- Rinse thoroughly with clean water.
- To dry, shake the parts to remove excess water. Allow to air dry thoroughly before reassembling.
- To reassemble, insert the Elbow Adapter into the opening on the **ComfortSeal®** Mask so it is horizontal when the mask is in an upright position.

Notes

- This product contains no latex (AER).

Cautions

- This product contains small parts when disassembled. Do not leave it unattended with children.

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